

ADULT CRASH CART

Adult Crash Cart

Front of cart- Arrest Board

Top of cart

- ✓ Clipboard
- ✓ Zoll- checklist
- ✓ Exam gloves- 1 box under defib
- ✓ ACLS and PALS algorithm cards
- ✓ Adult and Peds drug cards
- ✓ Crash cart checklist
- ✓ Lead II placement handout
- ✓ Resuscitation record (3) (code sheet)
- ✓ Zoll Defibrillator, appropriate pads for the defib
- ✓ Defib pads – plain gel (2)
- ✓ Zoll External pacer/hands off cable
- ✓ Sharps box
- ✓ Gloves (under defibrillator)

**First Supply To Expire
Located On IV Pole –
In a Sleeve
(back of cart)**



Cart Lock



PROCESS FOR EXCHANGE PROGRAM

Nursing is responsible :

✓ Call Distribution for cart exchange

✓ Remove defibrillator from used cart



✓ If laryngoscope or McGill forceps is used, send to Sterile Processing. For the laryngoscope, send the whole set (handle and all blades).

✓ Remove clipboard, Adult and Peds drug cards, ACLS and PALS algorithm cards, external pacer/hands off defib cable, checklists, resuscitation records, and Lead II placement diagram

✓ Remove any unique unit items from cart (such as internal defibrillator paddles)

✓ Replace defibrillator and unique supplies listed above to replacement cart

✓ Note expiration date on new cart (lock is applied by Distribution before sending to unit)

✓ Document exchange lock number and new expiration date on daily checklist

Adult Crash Cart

Right Side of Cart

Right Side of Cart

- ✓ Oxygen cylinder (check volume)
- ✓ Hand Resuscitator bag and face mask- adult
- ✓ Hand Resuscitator bag and face mask- Pedi Plus
- ✓ Hand Resuscitator bag and face mask- infant
- ✓ Sharps Container




Top of Cart

Drug Tray-Inside Drawer



Top of cart has a separate lock for medications

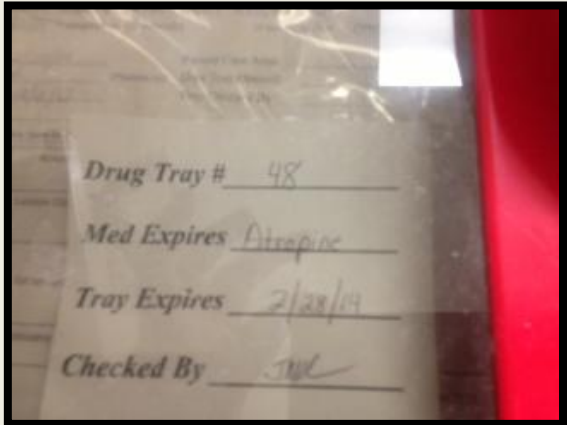

**Drug Tray- First Drug To Expire
Medication Expiration Date Located On Top
Of Drug Tray**

PROCESS FOR EXCHANGE PROGRAM- cont

Nursing is responsible :

- ✓ Send opened or expired drug tray to Pharmacy for replacement
- ✓ Sign for receipt of new drug tray
- ✓ Insert new sealed drug tray into top space of crash cart
- ✓ Apply new lock (supplied with replacement drug tray) and note number
- ✓ Document new lock number and new expiration date on daily checklist

Additional Nursing responsibilities:

- ✓ Doppler will be kept in secure location other than on the crash cart
- ✓ Nursing is responsible to assist Distribution in the Exchange process
- ✓ **Nursing is responsible to keep the outside of the cart clean**

Left Side of Adult Cart



Bin #1 BP Cuff/Steth (labeled)

- ✓ Numbered replacement locks
- ✓ Stethoscope
- ✓ BP Cuff adult (blue), thigh (burgundy)-disposable
- ✓ Gauge manometer for BP cuff
- ✓ Bulb inflation valve
- ✓ Adapter for W/A BP cuff
- ✓ Braslow Tape
- ✓ Flashlight (battery replace monthly)

Bin #2 IV Fluids (labeled)

- ✓ 250cc NS (2)

Bin#3 IV Fluids (labeled)

- ✓ NS 1 L (2)

Bin#4 Electrodes/Paper (labeled)

- ✓ Electrodes (2)
- ✓ Zoll EKG paper
- ✓ Goggles or glasses

Adult Crash Cart

Drawer 1- Airway Supplies(labeled)



- ✓ Syringe 10cc luer tip
- ✓ Endotracheal (subglottic) tubes
Sizes:6.5,7,7.5,8 (2 ea)
- ✓ Disposable stylet (2)
- ✓ Airways-Adult : small, medium, large (1ea)
- ✓ Epistick
- ✓ Oxygen tubing
- ✓ Non-rebreather mask
- ✓ Nasal cannula
- ✓ Laryngoscope handle and blade set
(battery/bulb)
- ✓ Yankauer
- ✓ O2 flowmeter w/ green nipple
- ✓ NS lavage 5cc pink
- ✓ 1"cloth tape
- ✓ Magill forceps
- ✓ Protective barrier wipes
- ✓ EtCO2adapter for R- Series Zoll adult/ped (2)
- ✓ CO2 Detector (2)
- ✓ LMA- Size1,1.5,2,3,4,5 (1 ea)
- ✓ Infant simple oxygen mask
- ✓ Peds simple oxygen mask

Adult Crash Cart

Drawer 2- IV Supplies(labeled)




- ✓ 30cc syringe luer tip
- ✓ Blood tubes-gold, mint green, purple and blue (2 each)
- ✓ Blood transfer device (2)
- ✓ Blood gas syringe (3)
- ✓ Blood gas kit (3)
- ✓ Tourniquet
- ✓ Saline flush pre-filled syringes (10)
- ✓ Sterile NS 30cc (2)
- ✓ PRN adapter (2)
- ✓ 1" silk tape
- ✓ Needles:1-1/2 inch-sizes 18,21,23 (5 each)
- ✓ Needles:1 inch size 25 (5)
- ✓ Blunt fill needle-(6)
- ✓ Syringes: size 1cc,5cc,10cc (5 each)
- ✓ 4 way stop cock
- ✓ Extension T set
- ✓ Chlorascrub pads
- ✓ Alcohol pads- for peds
- ✓ IV start kit (2)
- ✓ Cardiac/Spinal needle size 18, 20 (2 each)

- ✓ Tegadem 2"/4" (2 each)
- ✓ IV catheters: size 16,18,20,(4 each)22,24 (2 each)
- ✓ Butterfly: size 23,25 (2 each)
- ✓ Dry gauze dressing:4x4,2x2 (4 each)

Adult Crash Cart

Drawer 3- Pressure bag/IV tubing (labeled)

- ✓ 2" foam tape
- ✓ 4 way stopcock
- ✓ Pressure bag 1000cc
- ✓ IV tubing mini-drip (60gtt)
- ✓ IV tubing gravity set (20gtt) (3)
- ✓ IV pump tubing (3)

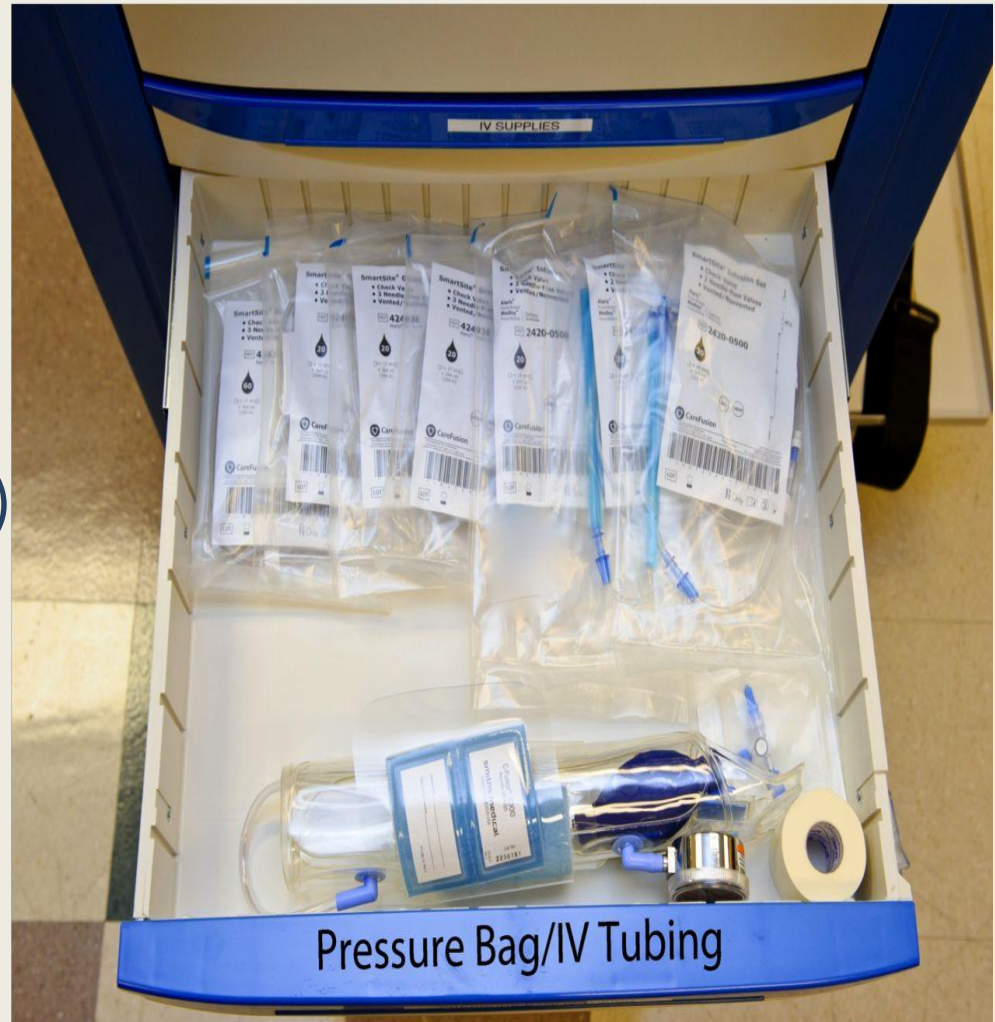

Adult Crash Cart

Drawer 4

Suction/Doppler/NGsupplies/Gloves/Drapes (labeled)



- ✓ Sterile drapes (2)
- ✓ Chloraprep swabs (6)
- ✓ Sterile gloves: sizes 7,7.5,8 (2 each)
- ✓ Doppler gel
- ✓ Suction regulator, canister, tubing (2)
- ✓ Suction catheters 8fr (2)
- ✓ Suction catheters: #14 clear, #14-16 (red rubber)
- ✓ 5 in 1 connector
- ✓ Lubricant jelly (4)
- ✓ Cath tip syringe 60cc
- ✓ Nasogastric tube: size 16, 18
- ✓ Doppler- required- store on unit

Adult Crash Cart

Bottom Shelf



- ✓ Multi lumen Central Vein Cath kit
- ✓ Trach Tray
- ✓ Zoll R Series defib/electrode pads
- ✓ Peds only R series