



Abridged Operating Instructions

KKX53120M-EN, Issue 3, 07/2012

1. Fit the sling to the patient (ensuring correct size is used) ensure Velcro strap is fastened securely and patient's arms are outside the sling.
2. Position the SARA PLUS in front of the patient, open the chassis legs if necessary, assist in placing patient's feet on the foot support with their knees touching the kneepad and apply castor brakes, allow patient to hold hand grips and rest their arms on the Arcrest.
3. Adjust height of Arcrest to its lowest position and adjust kneepad if necessary.
4. Attach the sling to the cord attachments and pull cords to eliminate slack.
5. Using the handset or dual control, raise the patient to a standing position.
6. Release brakes, close-up legs and transport patient to chair, bed or toilet.
7. Position the patient in front of chair, bed or toilet with the back of their legs touching the chair etc. and lower to a sitting position.
8. Release tension on the cords and remove cords from the sling.
9. Remove the SARA PLUS, taking care to remove the patients feet from the Foot support.
10. Remove the sling from around the patient.



Before operating the SARA PLUS and for full descriptions and instructions on Lifter operations, sling attachment safety warnings, battery charging and care of your SARA PLUS always refer to full operating instructions for your SARA PLUS.

CE
0086



Warning: The Pro-Active knee pad can be removed to allow knee or walking exercise. Ensure that when the knee pad is removed, the adjoining support bracket is always removed before starting therapy. When refitting, ensure the pad is reinserted and covers the support bracket. Failure to do so could result in serious injury.