

Ultralift 3500XH Quick Reference Guide



1 Hand Controller / Control Box

2 Emergency Stop

Push red emergency stop button located on base of lift

3 Spreader Bar Positioning Seated Lifting

Parallel to the patient's shoulders

4 Charging of Batteries

- Charge the lift regularly after use.
- The lift does not function during charging.
- Lift will not overcharge.

5 Spreader Bar Positioning Supine Lifting/Rolling

Spreader bar placement is perpendicular to the patient's shoulders

6 Emergency Lowering

Pulling upward on the red emergency lowering control initiates the lowering process

7 Locking the Wheels

Do not lock the wheels when lifting or lowering patients

8 Safety Clips

Ensure the safety clip on the spreader bar is in position after sling has been applied.

Ergonomics

- Never lift manually - let the lift do the lifting.
- Work calmly and methodically
- Don't strain your back! Work in a walking stance with your back straight.
- Use the bed's raising and lowering functions.
- Reduce the lever effect. Work close to the patient and the lift.

Simple Troubleshooting

Check to ensure that:

- Emergency stop is not pressed in.
- The battery is charged.
- The hand control is connected properly.

Cleaning

When necessary, clean the lift with a moist cloth, using common hard surface cleaners or disinfectants.