

PowerChart Touch: FetaLink+

FetaLink+ can be used on mobile devices to view fetal and maternal clinical waveform and patient data. Historical waveform data can also be viewed for the current monitored episode. FetaLink+ is a view-only application.

This handout will show how to use FetaLink+ on your iPhone or iPad.

Launch FetaLink+ ---p. 1	Additional Information Found within FetaLink+ ---p. 3
Viewing Patient Monitor ---p. 2	Information Found on Monitor List ---p.3

Launch FetaLink+



1. Select the FetaLink+ app, log on with your Cerner User Name and Password.
2. You will see all fetal monitors that are powered on. (The MFM clinic monitors are always powered on, but will not have a patient associated.) The list will be split between *Clinics* and *Hospital*. *Note: All monitors will appear under 'WCH'.

Clinics list will include monitors at MFM, Smiley, and Outpatient.

Hospital list will include monitors at L&D, OB Triage, Antepartum, AND the monitors at UH named: UH ICU and UH ED.



Viewing Patient Monitor

1. Once you tap on the patient's name within the list, the monitor strips will then appear full screen on your device.



The strip will display at actual time. The strip can be viewed for the past 72 hours plus the previous midnight (GMT) OR when the current patient was associated to the current device—whichever is shorter.

The time span can be viewed by *swiping* the strip to the *right*.

2. Charted Annotations can be viewed in a list form by tapping Annotations.

Annotations (197)

By selecting a specific annotation note on the list, you will be taken to view that time point on the strip.



3. Return to Monitor List by tapping the



menu. *Note: location of this menu may differ depending on the surface

Additional Information Found within FetaLink+

Besides the *Monitor* function of viewing the strips, there are other additional VIEW ONLY functions.



Review

Labor Assessment, Lab Results, Problem List, Allergies, and Vital Signs.


Orders

All active orders are viewable. This is a VIEW ONLY function, unable to place orders within FetaLink+

The carrot menu drop down on the Blue Banner Bar allows you to view Patient Demographics.



Information Found on Monitor List

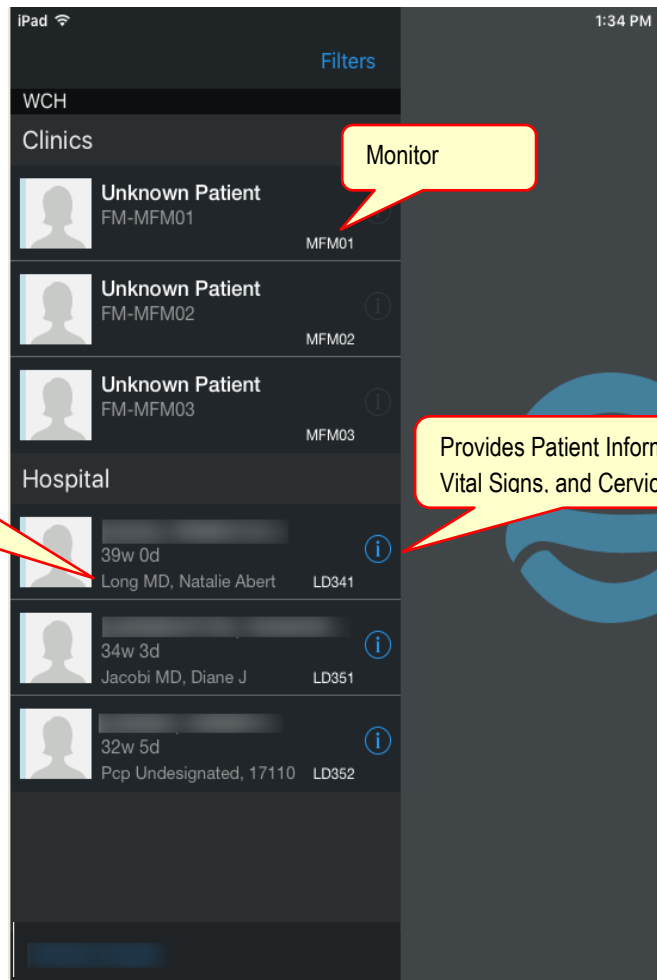

The Physician listed is the patient's identified **Primary Care Provider**.

Provides Patient Information, Vital Signs, and Cervical Exam.